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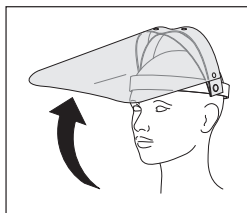
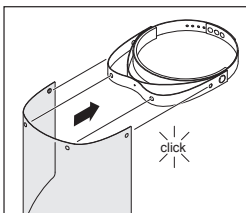
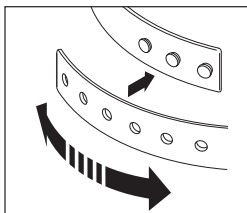
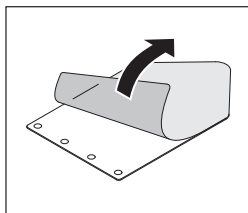
## FACE VISOR PRO

# 3433

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ECS European Certification Service GmbH,  
Hutfeldstrasse 50, 73430 Aalen, Germany  
Notified body no. 1883

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## EN Operating Instructions

This article complies with the regulation (EU) 2016/425 Category II on personal protective equipment (PPE), is type-tested and CE labelled. In order to safely handle this product, the user must have read and understood the operating instructions before using it for the first time.

### Use

This article protects against droplets and splashes of liquids. It reduces the risk of a droplet infection. Due to design considerations, complete protection against viruses, bacteria or other biological agents is only provided to a limited extent. Can be combined with face protection masks. Ensure proper disposal after use. Reusing it is up to the user's own discretion.

We recommend using the article for a maximum period of 2 years. The duration of use depends on various factors such as frequency of use, cleaning, storage and maintenance. Frequent checks and replacement of the visor film in case of scratches are recommended.

### Storage

Storage: in original packaging, in a dry environment. Storage condition: approx. 18 °C – 23 °C, approx. 55 % humidity.

### Duration of use

Do not use the article for more than 2 years after the date of purchase.

### Cleaning and Disinfection

With clear, lukewarm water and household washing-up liquid (no caustic solutions, acids or solvents). Avoid strong mechanical cleaning. The article can be disinfected with standard, commercially-available disinfectant.

### Risks

This article does not provide protection against any type of radiation (for instance, sunlight, UV, welding glare, IR). This eye protection equipment is only suitable for using at room temperature and is not fire-resistant. If the article is worn over corrective glasses, mechanical effects (for instance impacts) can be transferred, which represent a risk. Do not use the face shield if it is damaged, because this may influence sight and protection. If the eye protection equipment is damaged, it must be replaced (sharp edges can cause injuries). Materials that come into contact with the wearer's skin could cause allergic reactions in sensitive individuals.

### Accessories

Replacement foils for this face shield are available from DURABLE under the article number 343419 (10 pieces in the bag).

### Declaration of conformity

You will find the declaration of conformity at:

[www.durable.eu/3433CE](http://www.durable.eu/3433CE)

### Certification authority

ECS European Certification Service GmbH  
Huttfeldstrasse 50, 73430 Aalen, Germany

Notified body no. 1883

The article complies with the following regulations of the European Union: Regulation (EU) 2016/425 Personal Protective Equipment (PPE) and Directive 2001/95/EC General Product Safety, harmonised norm EN 166:2001 Category II

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