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Kontraindikationen

Do not use HydroTac transparent on patients with an intolerance to any of its ingredients, especially polyurethane, glycerin or acrylic based adhesive. Do not use on wounds which are clinically infected or on 3rd degree burns.

Product Disposal

To minimize the risk of potential infection hazards or environmental pollution, disposable components of HydroTac transparent should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

Contraindications

Do not use HydroTac transparent on patients with an intolerance to any of its ingredients, especially polyurethane, glycerin or acrylic based adhesive. Do not use on wounds which are clinically infected or on 3rd degree burns.

Possible adverse events

As an adhesive device, this product can have side effects, which do not necessarily occur for everybody.

General side effects of HydroTac transparent:

- In rare cases HydroTac transparent might cause local reactions (irritation, skin reactions e.g. redness, swelling)

In case you notice any effects not listed here, contact your doctor or pharmacist.

Special precautions

Medical assessment of the condition of the wound and the causes of any delay in wound healing are necessary before treating wounds with impaired healing. Treatment with HydroTac transparent cannot replace treatment of the cause of impaired healing.

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AT – PAUL HARTMANN Ges.p.H. - D-2355 Wiener Neudorf

CH – IVT HARTMANN AG - 8212 Neuhausen

FR**Instructions for use****Product description**

HydroTac transparent® ist ein transparenter hydrogel dressing consisting of hydrated polyurethane. The backing consists of a water vapor permeable polyurethane film which is impermeable to liquids and bacteria.

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Properties and mode of action

HydroTac transparent provides moisture and absorbs exudates during the phase of granulation and the phase of epithelialization.

HydroTac transparent is suitable for the treatment of dry or lightly exuding wounds in the granulation and epithelialization stages.

HydroTac transparent is transparent and provides an atraumatic dressing change. Because of the transparency, wound inspection is possible at any time without having to remove the dressing.

As HydroTac transparent comfort has a skin-friendly adhesive film border, no other materials are needed to secure the dressing.

Intended use / Indications

HydroTac transparent is suitable for the treatment of dry or lightly exuding wounds in the granulation and epithelialization stages.

HydroTac transparent is intended to be applied by wound care professionals.

Wound care professionals may delegate dressing changes to patients and/or their relatives (non-professionals under guidance of professionals).

Mode of application

Select a size of HydroTac transparent so that the wound pad overlaps the margins of the wound by approx. 1-2 cm. If required, the product HydroTac transparent may be cut to desired size, using sharp scissors only. Surrounding skin should be dry before applying HydroTac transparent comfort with adhesive border.

HydroTac transparent: Remove the blue protective layer following the numbering (1 & 2). Place the dressing with the gel-impregnated side on the wound and fix with suitable secondary dressing material.

HydroTac transparent comfort: Remove the blue protective layer following the numbering (1 & 2). Place the dressing with the gel-impregnated side on the wound and press down the adhesive edges firmly. Surrounding skin should be dry before applying HydroTac transparent comfort with adhesive border.

HydroTac transparent: Remove the blue protective layer following the numbering (1 & 2). Place the dressing with the gel-impregnated side on the wound and fix with suitable secondary dressing material.

HydroTac transparent comfort: Remove the blue protective layer following the numbering (1 & 2). Place the dressing with the gel-impregnated side on the wound and fix with suitable secondary dressing material.

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